

Cochlear™ Nucleus® 5 Sound Processor

Skitit® Instructions

Purchasing your Skitit


1. Visit Skitit.com and search for Cochlear Nucleus
2. Select Nucleus 5
3. Choose your design and place your order

Applying Skitit to your sound processor


1. Gently wipe the sound processor with a clean, dry, non-abrasive cloth
2. Slightly bend the Skitit adhesive card and remove the skin
3. Place the skin in the corresponding area of your sound processor
 - The Skitit card is cut to fit your sound processor, battery module, and sound processor cover(s)
4. Rub the surface to ensure the skin is secured

Your Skitit


Applying Skitit to your Standard sound processor cover

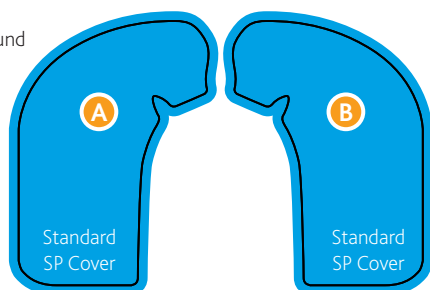


Applying Skitit directly to your sound processor

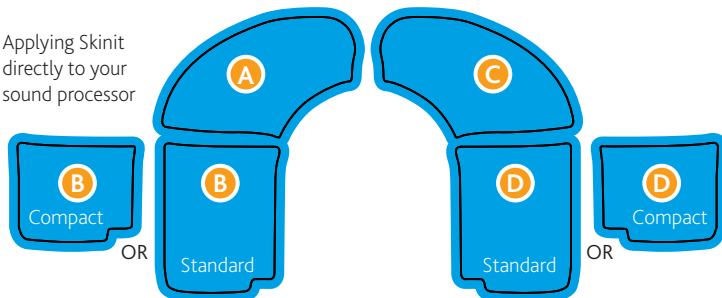

Applying Skitit to your compact sound processor cover

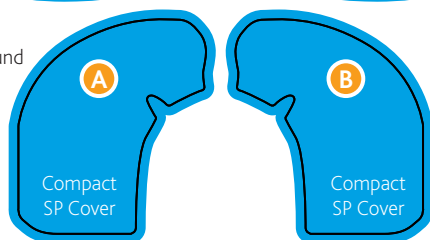

Your Nucleus 5 Sound Processor

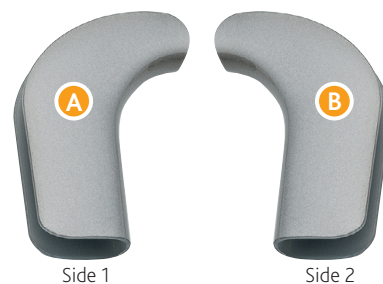

Skitit products are the property of Skitit, and therefore, all transactions and customer support are handled through Skitit.com. All orders are subject to Skitit terms and conditions. Cochlear cannot guarantee that Skitit skins will stay on your processor if it is exposed to water. Please remove your Skitit skins prior to returning your sound processor. Cochlear is not responsible for lost Skitit products.

Nucleus is a registered trademark of Cochlear Limited. Cochlear, and the elliptical logo are trademarks of Cochlear Limited. Skitit is a trademark of Skitit.com. All rights reserved.

FUN1473 ISS1 OCT11 © Cochlear Americas 2011