

Cochlear™ Nucleus® Freedom™ Sound Processor

Skinit® Instructions


Purchasing your Skinit

1. Visit Skinit.com and search for Cochlear Nucleus
2. Select Freedom
3. Choose your design and place your order

Applying Skinit to your sound processor

1. Gently wipe the sound processor with a clean, dry, non-abrasive cloth
2. Slightly bend the Skinit adhesive card and remove the skin
3. Place the skin in the corresponding area of your sound processor
 - The Skinit card is cut to fit your sound processor and BTE controller
4. Rub the surface to ensure the skin is secured

Your Skinit

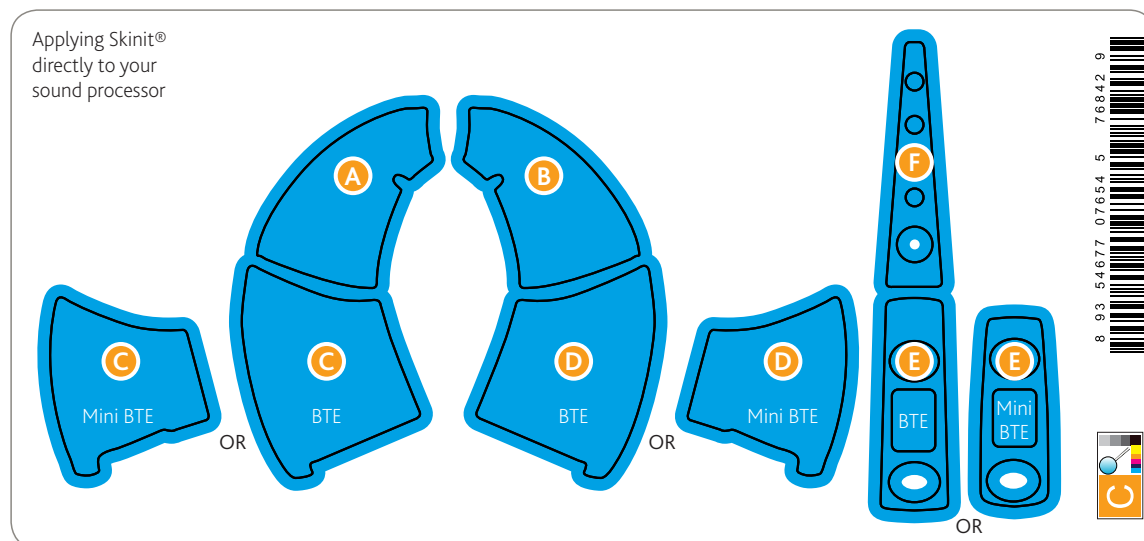

Your Freedom Sound Processor



Skinit products are the property of Skinit, and therefore, all transactions and customer support are handled through Skinit.com. All orders are subject to Skinit terms and conditions. Cochlear cannot guarantee that Skinit skins will stay on your processor if it is exposed to water. Please remove your Skinit skins prior to returning your sound processor. Cochlear is not responsible for lost Skinit products.

Nucleus is a registered trademark of Cochlear Limited. Cochlear, Freedom, and the elliptical logo are trademarks of Cochlear Limited. Skinit is a trademark of Skinit.com. All right reserved.