

Magnetic Resonance Imaging (MRI)

for patients with cochlear implants

A Nucleus® Freedom™ cochlear implant user can have an MRI with a field strength up to 1.5 Tesla as long as the internal magnet is removed through a simple surgical procedure.

The user must also remove the sound processor before entering a room where a MRI scanner is located.

If you require any additional information, please contact Cochlear Americas at **1 800 523 5798** or **www.cochlear.com**



Airport and other security personnel

Please be aware that this person has a small implanted hearing device and external sound processing equipment, which might set off airport security metal detection systems.

Without this equipment in place, the individual is completely deaf. Please give this person every consideration.

The implanted device is under the skin behind the passenger's ear and the external processing equipment can be worn over the ear, in a pocket, or pouch under clothing.

FUN961 ISS2 JUN09

